



Introducing Rinsendo

A revolutionary new method to clean root canals

An improvement in Endodontic therapy you will appreciate

Rinsendo is a highly effective and time efficient method of irrigating prepared root canals. Thorough debridement of root canals is necessary for successful endodontic treatment. The Rinsendo handpiece fully irrigates right to the apex. Germs and debris are effectively removed even under difficult anatomical conditions such as strong curvature and small diameters. Rinsendo supports successful endodontic treatment by facilitating the best possible conditions for long-term maintenance of the treated tooth.

Gentle, safe and effective treatment

Rinsendo replaces time consuming manual syringe irrigation. The ergonomic titanium handpiece accepts a specially designed single-use cannula with a 7mm long aperture. The long aperture eliminates blockage, enabling the irrigation solution to reach and disinfect all portions of the canal. A protective stopper guards against splatter and serves as a positioning device for the saliva ejector.

During operation, Rinsendo employs exclusive, patented pressure-suction technology to thoroughly flush the canal. In the pressure phase, 65 micro-liters of solution are automatically drawn from the attached syringe and aspirated into the canal. In the suction phase, the used solution and air are aspirated back. The pressure created by Rinsendo irrigation is lower than the pressure created by a syringe during manual irrigation.

Simple operation

Rinsendo is powered with dental compressed air. It is operated by the footswitch of your dental unit. A quick connect attaches Rinsendo to the handpiece tubing. Rinsendo is compatible with all irrigation solutions.



Prepared for treatment



Disinfection right to the apex



*Cutaway drawing showing
Pressure-Suction Technology*

**AIR
TECHNIQUES**
INC.



A revolutionary new method to clean root canals

If you are currently performing Endodontic procedures or are considering adding Root Canal Therapy to your practice you will value Rinsendo.



Specifications

Air Consumption:	0.70 CFM
Air Supply Pressure:	35 to 60 psi
Irrigation Speed:	6.2 (ml/min)
Pulsation Rate:	1.6 Hz
Dimensions (L x W x H):	6" x 0.9" x 1.4"
Weight:	3 ounces
Rinsendo Kit includes:	Rinsendo handpiece, Coupling, Disposable Cannulas (qty. 20), Protectors (qty. 20), Seal Kit with Silicone Lubricant, Disposable Syringes 5 ml (qty. 3), Instruction Manual
Sterilization:	Rinsendo is autoclavable at 134°C.
Warranty:	Rinsendo is warranted to be free from defects in material and workmanship from the date of installation for a period of 12 months.



Rinsendo handpiece with disposable cannulas and protectors



Rinsendo fits leading handpiece couplings



Rinsendo disposable cannula

Contact your authorized Air Techniques dealer or visit www.airtechniques.com for more information and to schedule a demonstration.



1295 Walt Whitman Road, Melville, New York 11747-3062 ● www.airtechniques.com
516-433-7676 ● Fax: 516-433-7684 ● Toll Free in the U.S. and Canada: 1-800-AIR-TECH ● Western Facility: 954-822-2899